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DIAQUICK Test to detect THC - cannabis, marijuana in a urine sample - For home use

Principle

Dialab's drug test is an extremely specific quick test to detect THC - cannabis, marijuana. The drug test is a proven testing method, used in many laboratories worldwide. The test's detection limits meet international standards. Using this test, you can detect the presence of THC in urine. Because of the test's high sensitivity, in the range of nanograms, in practice the test will detect any quantity of drug use. The drug test is not dangerous for the user when used correctly. Please follow the instructions for use of the test when carrying out the test and be aware of the fact that the urine samples tested may contain contagious materials. When testing unknown substances, the tested substance may contain substances harmful to health.

For your safety we recommend

- Read all the information in this leaflet before carrying out the test.
- Keep away from children.
- Do not eat, drink or smoke while carrying out the test.
- Do not use the test after its expiry date.
- Do not open the test bag before you are ready to begin the test.
- Do not reuse any component of the test.
- The used components of the test are to be disposed of in a garbage bin.
- Store in a dry place at a temperature of 2-30°C (36-86°F). Do not freeze.
- Do not use the test if the bag is torn or damaged.
- For external use only.
- After carrying out the test, clean well anything that has come into contact with the tested substance (and your hands).

Summary and explanation

Marijuana - THC, cannabis, hashish, grass

Marijuana is produced from the foliage of the cannabis plant and smoking is the main method of consumption. This may cause slight addiction; the symptoms, which continue for 2-4 hours, include disorientation, dizziness and psychosis. The period during which marijuana can be detected in urine is 1-14 days (in chronic users : up to 80 days).

Storage and stability

The Diaquick drug test is to be stored at a temperature of 2-30°C in the original sealed bag. The expiry date is determined in nominal laboratory conditions. The kit can be stored at room temperature or refrigerated (2-30°C). The testing stick is stable until the expiry date printed on the sealed bag. Keep the testing stick in the sealed bag until using. Do not freeze. Do not use after the expiry date.

Collecting and preparing the samples

Collect the urine samples in a clean dry container. Urine collected at any time of day can be used.

Storing the samples

The urine samples can be stored at a temperature of 2-8°C for up to 48 hours before carrying out the test.

Materials supplied

- Testing stick.
- Information leaflet.

Materials required that are not supplied

- Container for collecting urine (clean and dry).
- Clock for measuring time.

Instructions for use

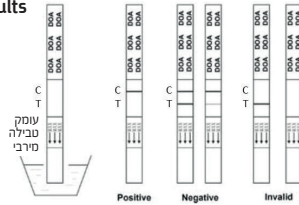
Before carrying out the test let the testing stick and the urine sample reach room temperature (15-30°C).

- Bring the bag to room temperature before opening it. Remove the testing stick from the sealed bag and use it as soon as possible.
- Dip the testing stick vertically into the urine sample for 10-15 seconds, with the arrows pointing towards the urine. Do not allow the urine sample to come into contact with the area of reading the results when dipping the testing stick into the urine sample. Avoid dipping the stick

deeper than the "max" sign marked on it and avoid any contact of the urine sample with the result area.

- Place the testing stick on a flat surface, begin to measure the time and wait for the appearance of a red line or lines. Read the result after 5 minutes. The results are not to be interpreted after more than 10 minutes.

Interpreting the results

Negative: A colored line in the control area (C) and a colored line in the test area (T) for THC, is a negative result. This means that the concentration of the drug in the urine is lower than the threshold level defined for this specific drug.

* Comment: the hue of the color in the test area (T) may change, however the result will be regarded as negative even if there is a faint pink line.

Positive: A colored line in the control area (C) without a colored line in the test area (T) for THC, indicates a positive result. This result means that the concentration of the drug in the urine sample is greater than the defined threshold level.

Invalid: No control line appears. Insufficient test volume or defective testing methods are the most probable reasons for non-appearance of a control line. Learn how to carry out the test and repeat it using a new testing stick. If the problem recurs, immediately stop using the tests from the same batch and contact your local supplier.

Quality control

Control of procedures included in the test itself. A red line that appears in the control area (C) is regarded as an internal procedure control. It indicates adequate volume of the sample, correct activity of the membrane and correct testing technique.

Limitations

Limitations of the testing process
This test may give incorrect positive or negative results. Invalid results are mostly caused by an insufficient sampling quantity or by carrying out the instructions inaccurately. Read the instructions and repeat the test using a new kit. If the problem recurs, immediately stop using the tests from the same batch and contact your local supplier. Do not use a positive result to determine the concentration of the drug in the sample.


A negative test result does not necessarily mean that the sample is free of drugs. There may be drugs in the sample below the test's threshold concentration, or the sample may contain disrupting substances. Do not use this test as a basis for therapeutic means.

Operational characteristics

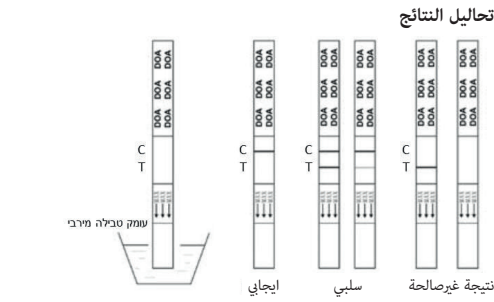Cross reactions
Research has been carried out to determine cross reactions of the sample with different substances, to which THC was added.

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